AXIOM

AS

ASP

USER INSTRUCTION MANUAL AND WARRANTY
MANUAL DE INSTRUCCIONES Y GARANTÍA

Table of Contents

IMPORTANT CONSUMER INFORMATION

| | | |
|----|------------------------------|---|
| A. | Intended Use..... | 2 |
| B. | Your Safety | 3 |
| C. | Signal Words..... | 3 |
| D. | Symbol Glossary | 4 |
| E. | Check Your Skin Daily | 5 |
| F. | General Warnings..... | 5 |
| G. | Protecting Your Cushion..... | 6 |
| H. | Obstructions..... | 6 |
| I. | Cover Orientation | 6 |

SET-UP, ADJUSTMENT AND CARE

| | | |
|----|---|----|
| A. | Fitting Your Cushion and Wheelchair..... | 7 |
| B. | Maintenance..... | 8 |
| C. | Cleaning | 9 |
| D. | Reassembling Your Axiom Cushion After Cleaning..... | 11 |
| E. | Axiom AS and Axiom ASP - Bottoming Out | 12 |

| | |
|---|-----------|
| KI MOBILITY LIMITED WARRANTY | 13 |
|---|-----------|

Axiom AS & Axiom ASP

IMPORTANT CONSUMER INFORMATION


NOTE: This manual contains instructions for use for the Ki Mobility Axiom AS and Axiom ASP cushions. This information must be provided to the consumer of this product. Do not remove this manual before delivery to the consumer of this product.

Supplier: This manual must be given to the consumer of this product.

Consumer: Read the entire owner's manual prior to using this product and keep on-hand for future reference.

NOTE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact the supplier or Ki Mobility for further instructions.

Cushion Features

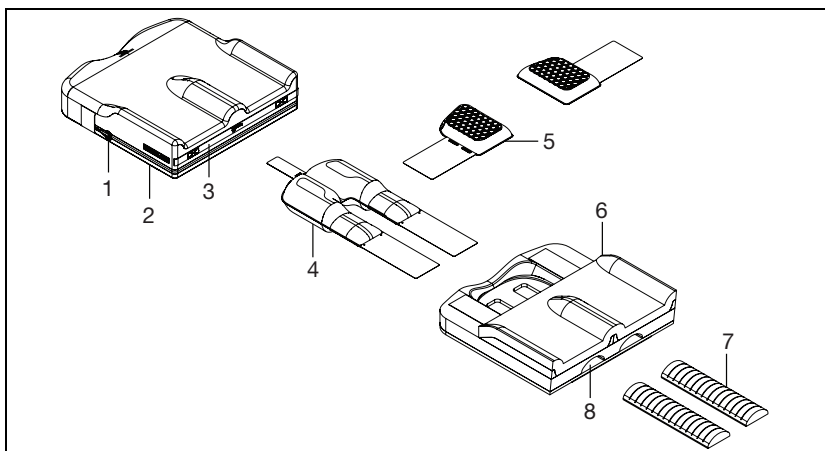| | |
|---|------------------------|
| 1. Water Resistant Zipper and Zipper Pull | 2. 4-Way Stretch Cover |
| 3. Front Handle | 4. Fluid Bladder |
| 5. Fluid Bladders | 6. Cushion Base |
| 7. Reservoir Plugs | 8. Reservoirs |

MAXIMUM USER WEIGHT: 350 lbs (158 kilos)

A. Intended Use

Ki Mobility Wheelchair Cushions are intended to provide postural support, pressure distribution and improved sitting tolerance.

B. Your Safety

Any serious adverse events or injuries related to the use of your product or its accessories must be reported immediately if the incident directly or indirectly leads to serious health decline or death of the user or other person. Report any serious events and/or injuries to Ki Mobility and, if required by local regulation, the competent health authority where the user and/or other person legally reside.

Do not use this product without first reading this entire manual. BEFORE riding, you should be trained in the safe use of this product by an Assistive Technology Professional (ATP) or other competent clinical or technical professional. Ki Mobility manufactures many different products that might meet your needs. The recognized best practice for selecting a product is to consult with an ATP or other competent technical professional and an experienced clinical professional such as a physical therapist, occupational therapist or physician. Final selection of the type of product, options and adjustments rests solely with you and your technical and clinical professionals. The options you choose, and the set-up and adjustment of the product have a direct impact on its performance, stability and its ability to meet your needs.

C. Signal Words

Within this manual you will find what are referred to as “Signal” words. These words are used to identify and convey the severity of varying hazards. Before using this chair you, and each person who may assist you, should read this entire manual. Please note the Signal word and consider any notes, cautions or warnings. Make sure to follow all instructions and use your chair safely. The Signal word refers to a hazard or unsafe practice that may cause severe injury or death to you or to other persons. The “Warnings” are in three main categories, as follows:

NOTE – Note indicates a potentially hazardous situation which, if not avoided, could result in a decline in the performance of the chair or damage to your wheelchair.

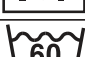
CAUTION: Caution indicates a potentially hazardous situation which, if not avoided, could result in damage to your wheelchair and injury to you.

WARNING: Warning indicates a potentially hazardous situation which, if not avoided, could result in serious injury or death.

These signal words will be placed throughout the manual, where appropriate to highlight the hazardous situation. Refer to Section G. for hazardous situations that will apply to the general use of this product.

D. Symbol Glossary

| SYMBOL | DEFINITION |
|---|---|
|  | Indicates the medical device manufacturer. |
|  | Date of manufacture (YEAR/MM/DD). |
|  | Indicates the need for the user to consult an instruction for safety reasons such as cautions and warnings. If presented on the medical device or packaging, it indicates the need for the user to consult the instructions for safety reasons such as cautions and warnings. |
|  | Indicates the item is a medical device. |
|  | Indicates the manufacturer's serial number so that a specific medical device can be identified. |
|  | Indicates the authorized representative in the European Community. |
|  | Indicates the manufacturer's declaration that the product meets the requirements of the applicable EC directives. |
|  | Indicates the need for the user to consult the instructions for use. |
|  | Indicates the need for the user to consult the listed website for instructions for use in an electronic format. |
|  | Indicates a website where a user may obtain additional information about the medical product. |
|  | Indicates a specified maximum weight limit (lbs/kg). |
|  | Indicates a transit securement point. |
|  | Indicates a potential pinch point. |
|  | Indicates the entity importing the medical device into the locale. |
|  | Indicates the manufacturer's batch code so that the batch or lot can be identified. |
|  | Machine Wash, Hot |
|  | Do Not Bleach |
|  | Line dry or tumble, no heat |
|  | Do Not Iron |
|  | Do Not Dry Clean |

E. Check Your Skin Daily

These Axiom Seating and Positioning products are designed to improve pressure distribution and increase sitting tolerance. Whether the cushion provided will meet your individual needs cannot be assured in advance. The integrity of your skin can be affected by many aspects of your daily life and medical condition, including the use of this product. Be sure to follow any skin care regimens established by your clinician.

Users of this product should make sure their skin is inspected daily for changes. If you see changes, discontinue the use of this product immediately and consult with your clinician. A key clinical indicator of tissue breakdown is skin redness. If your skin develops redness, discontinue the use of this product immediately and consult your clinician. Be especially careful to inspect the areas related to the bony prominences of the pelvis such as your trochanters and ischial tuberosities.

WARNING

WARNING: Failure to perform proper skin inspection or continuing to use a cushion after skin changes are present can affect your skin integrity, resulting in serious injury or death.

F. General Warnings

WARNING

WARNING: Do not use or install this cushion without first reading and understanding this manual. Please contact your authorized Ki Mobility supplier or your clinician if you are unable to understand the Warnings and Instructions. Failure to do so could result in serious injury or death.

WARNING

WARNING: The adhesive on hook and loop closures (commonly known as "Velcro") may degrade over time and reduce its ability to secure the cushion. This could cause you to fall from your wheelchair, resulting in serious injury.

WARNING

WARNING: The installation of a cushion on a wheelchair could affect the center of gravity of the wheelchair. Changes in your center of gravity may affect your stability in your wheelchair, resulting in tipping over or falling from your wheelchair which may result in serious injury. Always review the instructions for use of your wheelchair to see if changes to the wheelchair may be needed to provide sufficient stability after adding a cushion.

WARNING

WARNING: The integrity of your skin can be effected by many aspects of your daily life and medical condition, including the use of this product. Be sure to follow any skin care regimens established by your clinician. Consumers of this product should make sure their skin is inspected routinely for changes as directed by their clinician. Failure to do so could result in serious injury or death.

G. Protecting Your Cushion

Avoid prolonged exposure to adverse environmental conditions. This may include excessive exposure to sunlight and temperatures, either hot or cold. This can result in the degradation of the performance of your cushions.

WARNING: Failure to protect your cushion may lead to changes that can affect your skin integrity, resulting in serious injury or death.

Although your cushion has been manufactured in accordance with its labeled flammability standards, you should avoid exposing your cushion to open flames or extreme heat sources such as a burning cigarette. This may cause your cushion to ignite or emit harmful gases.

Failure to protect your cushion from extreme heat may cause your cushion to ignite or emit harmful gases, resulting in serious injury or death.

H. Obstructions

To ensure your cushion works effectively, avoid placing obstructions between your bottom and your cushion. Obstructions may include, but are not limited to:

- Keys or other items in pockets
- Lift slings
- Underpads or Chucks


Obstructions will lessen the effectiveness of the product, potentially causing pressure or high pressure points.

WARNING: Pressure points can affect your skin integrity, resulting in serious injury or death.

I. Cover Orientation

Your cushion was designed to be sat on in a specific orientation and it must be placed properly in your wheelchair with the cover oriented correctly with the foam base. It is labeled to indicate the proper orientation.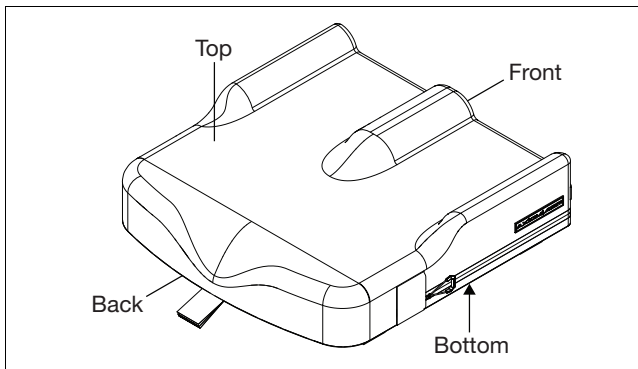


WARNING: Failure to properly orient the cushion and cover can affect your skin integrity, resulting in serious injury or death.

Set-Up, Adjustment and Care

A. Fitting Your Cushion and Wheelchair

1. Initial Cushion Set-Up Without User in Seated Position

- In determining the right cushion for you, ensure the dimensions are appropriate considering the width and length of your wheelchair sling or seat pan. If you are unsure, consult your authorized Ki Mobility supplier or your clinician.
- Place the cushion in the chair with the material identification tags facing the rear of the chair and the cushion handle oriented toward the front of the chair. The non-skid texture should be facing the seat sling or seat pan on your wheelchair.
- It is important that during normal use the cushion is retained in the wheelchair. It is also important that the cushion is not displaced during transfers that might cause you to slip or fall. Your cushion has hook and loop closures and a non-skid bottom to facilitate retention in a wheelchair. Ensure the cushion hook and loop closures align and mate with the wheelchair hook and loop closures. If your wheelchair sling or seat pan does not have hook and loop closures, or they do not align with the cushion, you should consult your authorized Ki Mobility supplier or your clinician to ensure the cushion is appropriate.

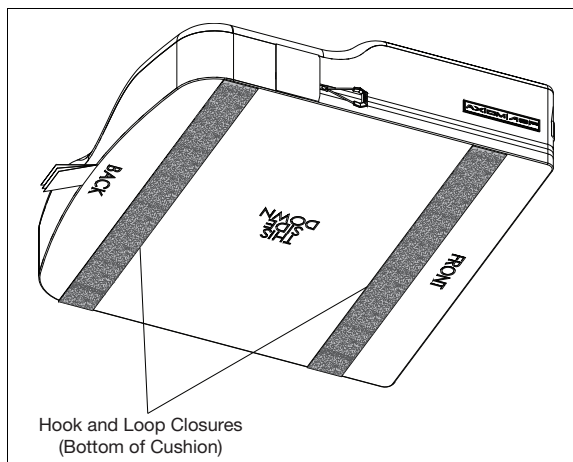⚠ WARNING ⚠

WARNING: Failure to properly select a cushion that fits your wheelchair can affect your skin integrity which can result in serious injury or death.

⚠ WARNING ⚠

WARNING: Failure to properly orient the cushion and cover can affect your skin integrity which can result in serious injury or death.

⚠ WARNING ⚠

WARNING: Failure to properly select the cushion and cover that can be retained in the wheelchair can cause you to fall from your wheelchair, resulting in serious injury.

⚠ WARNING ⚠

WARNING: Avoid using the cushion on overly stretched seat sling upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result. This can affect your skin integrity, resulting in serious injury or death.

A. Fitting Your Cushion and Wheelchair (Continued)

2. Wheelchair Set-Up and Adjustment for your Cushion
 - a. Ensure hips are placed properly in the back of the wheelchair and cushion as directed by your authorized Ki Mobility supplier or your clinician.
 - b. Ensure your footrests are adjusted to the proper height, as directed by your authorized Ki Mobility supplier or your clinician.
 - c. Ensure your back support is properly positioned and adjusted for your seated posture, as directed by your authorized Ki Mobility supplier or your clinician.

WARNING: Failure to ensure that the wheelchair is properly adjusted for use with the cushion can affect your skin integrity, resulting in serious injury or death.

B. Maintenance

Regular cleaning and maintenance may help extend the life of your cushion. You should inspect your cushion weekly. During cleaning, component inspection is recommended.

1. Check Cover for Tears and Excessive Wear
 - a. Remove cover and check for abnormalities (inside and out).
 - b. Check the foam base for breakdown or degradation.
 - a. Tears deeper than .25" (6.35 mm)
 - b. Loose granules of foam
 - c. Permanent changes to the unloaded foam shape, such as flattening.
 - c. Inspect the fluid bladder for tears or punctures. Inspect the area around the bladder and the cover for the presence of a thick, white fluid which will indicate the presence of a leak.

WARNING: If you find any abnormalities, such as the foam breaking down, discontinue the use of this product immediately and consult with your authorized Ki Mobility supplier or clinician. Failure to do so can affect your skin integrity, resulting in serious injury or death.

C. Cleaning

WARNING: Cleaning, deodorizing and sanitizing, even using the method described here, can accelerate the natural degradation of the foam. The effect will vary depending on the chemicals used. Verify that the degradation has not progressed to the point the cushion should not be used, according to the instructions for use provided with the product, failure to do so can result in serious injury.

| Care & Use Instructions | |
|---|-----------------------------|
|  | Machine Wash, Hot |
|  | Do Not Bleach |
|  | Line dry or tumble, no heat |
|  | Do Not Iron |
|  | Do Not Dry Clean |

1. Cleaning the Cover

NOTE: Drying with heat may cause shrinking that affects the covers fit and function.

- Remove the cover from the foam base and zip closed.
- Machine wash in hot water and tumble dry - no heat. Do not iron. Note - Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.
- To refit cover after washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base. Note - If utilized in an institutional setting, write the residents name with a permanent marker on the content label of the cover. This will ensure the return of the correct cover to its matching cushion after washing.

2. Cleaning or Deodorizing the Foam Base

NOTE: Do not immerse your foam base in any solution.

- Fill a spray bottle with detergent or deodorizer. Verify that the labeling for the manufacturer of the detergent or deodorizing indicates it is appropriate for this use. Follow the manufacturer's labeling for proper solution concentration and use.
- Lay the foam base in the bathtub or on a surface that can get wet.
- Spray the entire surface of the foam base including the sides. Flip it over and repeat on the bottom. Once you've finished spraying, let the solution set for 20 to 30 minutes.
- Wipe the solution off with a damp washcloth. Do not spray with water.
- Allow the material to air dry but avoid direct sunlight. Depending on how wet your foam base is, it may take 24 hours to several days to completely dry. Verify that no solution remains in the foam base before using.

C. Cleaning (Continued)

3. Sanitizing the Foam Base

NOTE: Do not immerse your foam base in any solution.

- a. Fill a spray bottle with a 70% or greater alcohol solution.
- b. Remove the fluid bladders and reservoir plugs.
- c. Lay the foam base in the bathtub or on a surface that can get wet.
- d. Spray the entire surface of the foam base including the sides. Flip it over and repeat on the bottom. Once you've finished spraying, let the solution set for 20 to 30 minutes.
- e. Wipe the solution off with a damp wash cloth. Do not spray with water.
- f. Allow the material to air dry but avoid direct sunlight. Depending on how wet your foam base is, it may take 24 hours to several days to completely dry. Verify that no solution remains in the foam base before using.

4. Cleaning or Deodorizing Fluid Pads

NOTE: Never immerse the fluid pads in any solution. Never clean with abrasive materials.

- a. Remove the cover from the foam base.
- b. Remove fluid pads from the inner cover.
- c. Fill a spray bottle with detergent or deodorizer. Verify that the labeling for the manufacturer of the detergent or deodorizer indicates that it is appropriate for this use. Follow the manufacturer's labeling for proper solution concentration and use.
- d. Spray and wipe the bladders with a clean dry cloth.
- e. Make sure the bladders are completely dry before reassembly.


D. Reassembling Your Axiom Cushion After Cleaning

WARNING

WARNING: Place the Axiom AS and Axiom ASP fluid insert into the well of the cushion base ensuring position is aligned. A misalignment or creation of an edge could cause an unwanted pressure point, which could lead to injury.

1. After Proper Cleaning and Maintenance
 - a. Install the trochanteric fluid inserts by mating the hook and loop closure sections from the fluid insert onto the hook and loop closure of the foam base as shown in image below.
 - b. Install the ischial fluid insert by mating the hook and loop closure sections from the fluid insert onto the hook and loop closure of the foam base as show in image below.
 - c. Reinstall the reservoir plugs so that the end of the plug is flush with the front of the cushion.
2. Install base with fluid insert into the outer cover.
3. Close the zipper on your cushion ensuring that the zipper-pull ends up closed with the pull tucked into the zipper pocket in the cover.

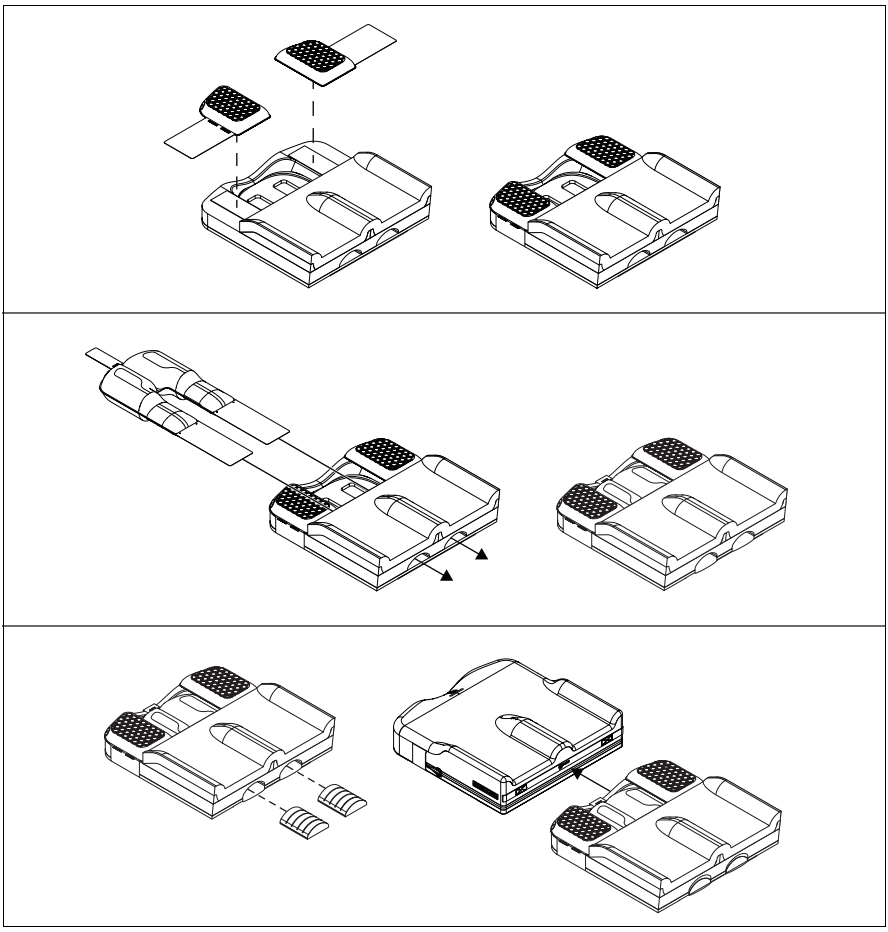E. Axiom AS and Axiom ASP - Bottoming Out

WARNING: If bottoming out is detected or you are unsure whether you are bottoming, discontinue the use of this product immediately and consult with your authorized Ki Mobility supplier or your clinician. Failure to do so can affect your skin integrity, resulting in serious injury or death.

Bottoming out may occur on an Axiom AS or Axiom ASP cushion if you displace all the fluid underneath your pelvic bones or buttocks, which would leave you sitting on the foam base. It sometimes occurs on very thin individuals, people using recliner wheelchairs, people who have lost weight or have other changes in body size or shape, or people who slouch when sitting.

To check for bottoming out, first make sure all other instructions for fitting the cushion and wheelchair are complete. Then sit on the cushion, with the cover on for five to ten minutes. Transfer up and off the cushion (or have someone help you transfer), trying not to disturb the fluid underneath you. Fold the back portion of the cover forward over the front, exposing the fluid pad. Push down in the depressions on the pad where your ischial tuberosities (seat bones) and coccyx (tailbone) were. You should have to push through at least .5" (12.5 mm) of fluid before you feel the firm cushion base below.

Changes to the cushion and changes to your body occur over time and you should check for bottoming out at least monthly.

KI MOBILITY LIMITED WARRANTY

The expected life of the product is 5 years.

A. For 24 Months

Ki Mobility warrants the cushion to be free from defects in materials and workmanship for a period of 24 months. This limited warranty applies to:

1. Axiom AS/ASP Cushion Base & Axiom AS/ASP Fluid Pads

For a period of 24 months (2 years) from the date of purchase, provided normal use, should a defect in materials or workmanship occur, Ki Mobility will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears, or burns.

B. For 6 Months

Ki Mobility warrants the cushion to be free from defects in materials and workmanship for a period of 6 months. This limited warranty applies to:

1. Axiom AS/ASP Removable Cover

For a period of 6 months from the date of purchase, provided normal use, should a defect in materials or workmanship occur, Ki Mobility will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns.

C. Limitations

1. We do not warrant:
 - a. Damage from neglect, accident, misuse, or from improper installation or repair.
 - b. Products modified without Ki Mobility's express written consent.
 - c. Damage from exceeding the weight limit.
2. This warranty is VOID if the original cushion tag is removed or altered.
3. This warranty applies in the USA only. Check with your supplier to find out if international warranties apply.
4. This warranty is not transferable and only applies to the first consumer purchase of this product through an authorized Ki Mobility supplier.

D. What We Will Do

Our sole liability is to repair or replace covered parts. This is the exclusive remedy for consequential damages.

E. What You Must Do

1. Obtain from us, while this warranty is in effect, prior approval for return or repair of covered parts.
2. Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

F. Notice to Consumer

1. If allowed by law, this warranty is in place of any other warranty (written or oral, express or implied, including a warranty of merchantability, or fitness for a particular purpose).
2. This warranty gives you certain legal rights. You may also have other rights that vary from state to state.

Record your lot number (located on cushion tags) here for future reference:

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Etac Supply Center AB